

DO YOU HAVE WI-FI?



Medtronic recommends you set up Wi-Fi to ensure that you have a more robust connection and are able to receive software and security updates.



The MyCareLink Relay™ Home Communicator will automatically default to a cellular connection if you do not use Wi-Fi. Setting up Wi-Fi is not required.




If you need assistance, please call 1-866-470-7709 (7 a.m.-7 p.m. CT, Monday-Friday)




Before starting up, you will need to have your Wi-Fi network name and password on hand.


- 1** Plug in the power cord.
If the communicator is already plugged in, unplug it and reconnect.

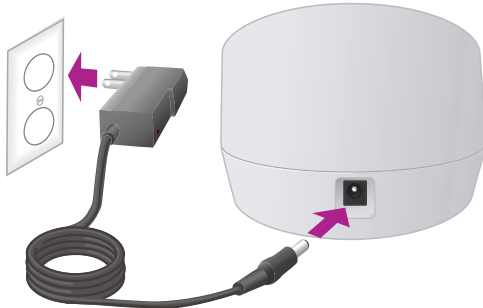

- 2** Open the wireless network connection window on your computer and select Medtronic. Please note that it may take a few minutes for the Medtronic Wi-Fi network to appear.

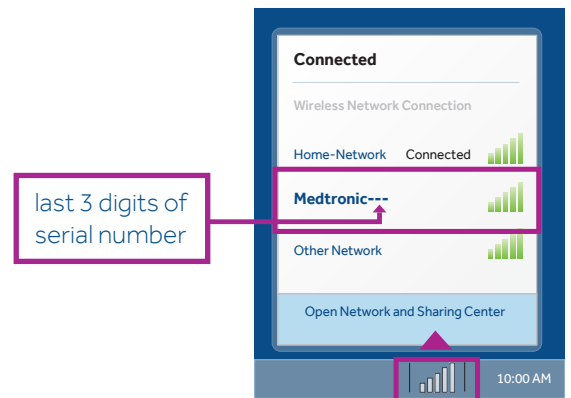

- 3** When prompted, enter the password from the communicator label (found on the bottom of the communicator).

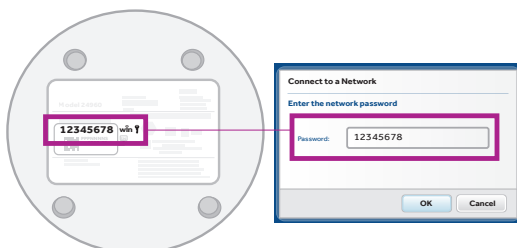


- 4** Open an internet browser and type in "<http://www.MCLRelayWiFi.com>" and hit enter. Be sure to include "http://".




Instructions continued on back →

Medtronic

5 Select a language and click **GET STARTED**.

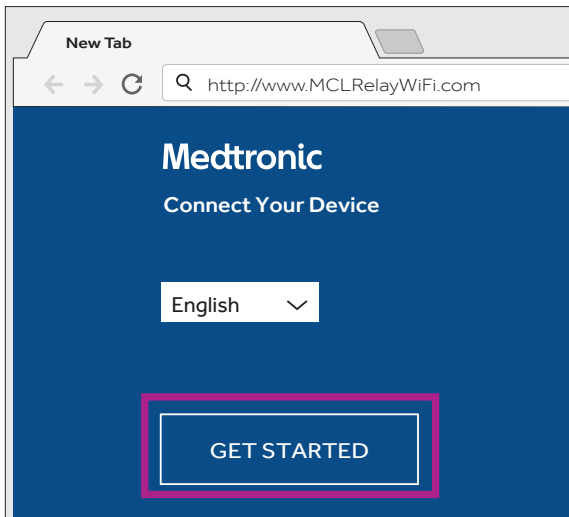

6 Select your home Wi-Fi network and press **CONTINUE**.

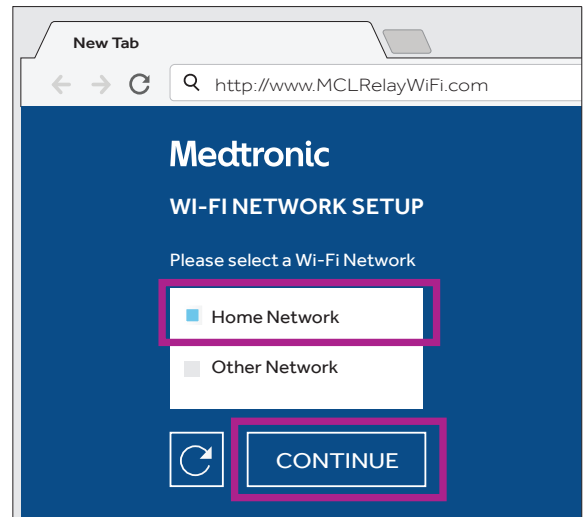

7 Enter your home Wi-Fi password and press **CONTINUE**.



8 Once you see the screen below, the communicator has successfully been connected to Wi-Fi!

